

▲
OPEN








HMC GROUP **REF IN01**

pH indicator strips

pH strips to indicate the pH level
of human gastric aspirate


Quantity: 200 Strips

LOT

HMC Premedical S.p.A.
 Via Bosco 1/3
 41037 Mirandola (MO) – Italy
 tel.: +39 0535 22704 fax.: +39 0535 609546

Distributor:
Medicina Ltd.
 Units 2-4, Rivington View Business Park,
 Station Road, Solton BL6 5BN UK


part of HMC Group


SMALTIRE COME RIFIUTO OSPEDALIERO
DISPOSE OF AS HOSPITAL WASTE

UDI Barcode here to include Lot & Expiry date

HMC GROUP **medicina**
part of HMC Group

pH indicator strips

Place a small amount of aspirate on the reactive section. Compare reactive section colour immediately with indication strip colours

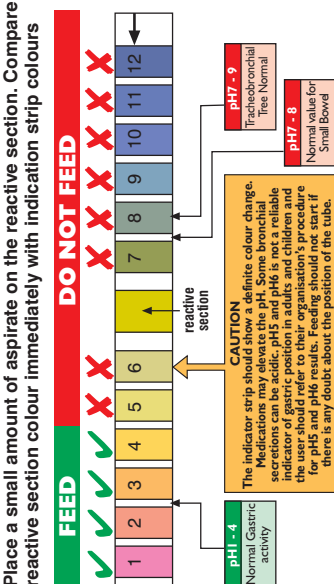

CAUTION
The indicator strip should show a definite colour change. Medications may elevate the pH. Some bronchial secretions can be acidic. pH 5 and pH 6 is not a reliable indicator of pH. In addition, if the aspirate is contaminated with these should not be used. If there is any doubt about the position of the tube, there is any doubt about the position of the tube.

STORE IN A COOL DRY PLACE AWAY FROM DIRECT SUNLIGHT

CAUTION: These position guidelines are not meant to replace the healthcare professionals' clinical judgement and should be used in conjunction with other clinical signs, such as coughing, retching and increased respiratory distress.

IN01LBL006 Rev.00_5/21

Description: IN01 Box Label			
Code: IN01LBL006		Size: W 63mm x H 191mm	
Colours: Full Colour		Date created: 25th May 2021	
Reason for artwork commission/revision: Change of Manufacturer and MDR			
Task	Name	Signature	Date
Proofed by			
Reviewed by			
Approved by			
APPROVALSBOX001			



REF IN01

pH indicator strips

LOT



200 x 200 strips

SMALTIRE COME RIFIUTO OSPEDALIERO
DISPOSE OF AS HOSPITAL WASTE



Consult instructions for use



Keep dry



Keep away from sunlight



Do not reuse



Caution



Medical Device



Manufacturer



HMC Premedical S.p.A.

Via Bosco 1/3
41037 Mirandola (MO) – Italy
tel.: +39 0535 22704 fax.: +39 0535 609546

Distributor



Medicina Ltd.

Units 2-4 Rivington View Business Park
Station Road, Bolton, BL6 5BN UK.

UDI Barcode here to include Lot & Expiry date

IN01CTL006 Rev.00_05/21

Description: IN01 Carton Label			
Code: IN01CTL006		Size: W 199.6mm x H 143.5mm	
Colours: Black		Date created: 25th May 2021	
Reason for artwork commission/revision: Change of manufacturer and MDR			
Task	Name	Signature	Date
Proofed by			
Reviewed by			
Approved by			
APPROVALSBOX001			