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Best Quality made by a German Manufacturer

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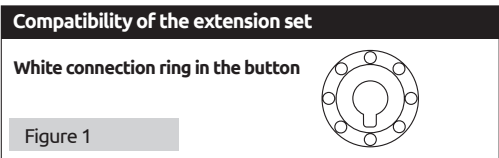
**en** danumed® Extension Set for Gastrostomy Button

#### Indikation

This/enteral extension set may only be used for enteral feeding and/or to administer medications via the stomach and/or for decompression. It may only be used on patients who have a low profile gastrostomy button (button) inserted. It serves to connect the button and feeding set or enteral syringes. **The enteral extension set may only be used on one patient (single-patient product) and must be replaced latest after 14 days.**


#### Compatibility

The button connector on this extension set is only compatible with buttons with a white connection ring as shown in Figure 1.

Configurations	
This enteral extension set is available in a sterile and non-sterile version, with tube lengths of 05 cm, 30 cm and 60 cm and in the following configurations:	

Feeding connector	Button connector	
	right angled	straight
	standard bore tube	wide bore tube
	Y-ENFit® ENSwivel	-
	standard bore tube	-
	ENFit® ENSwivel	-
	-	wide bore tube
	Bolus connector to connect a catheter tip syringe, in particular for Bolus feeding	

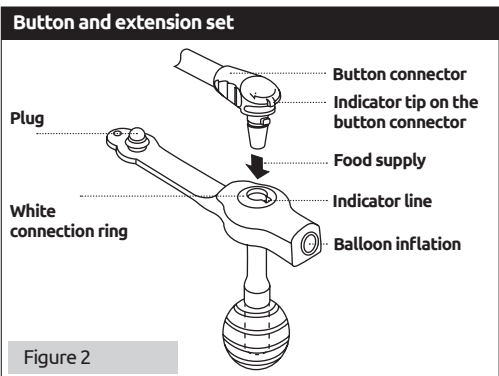- Instructions for use**
- Wash hands thoroughly.
  - Check the button for proper fit.
  - Carefully pull the plug on the button out of the feeding port so the non-return valve and the white connecting ring become visible.
  - Connect the button connector to the button. To do this, align the indicator tip on the button connector and the indicator line on the button over each other, lightly press down on the button connector and lock it into place by rotating it clockwise 1/4 to 1/2 of a turn. See also Figure 2.
  - Make sure it is securely locked to prevent unwanted leakage.
  - The suitability of the medication with respect to grinding them up and interactions with the materials must always be clarified with the treating physician and/or pharmacy.
  - **Never** mix or administer medications with a nutrient solution!
  - After administering the enteral food and/or administering medication and/or decompression, rise the enteral extension set and the button with 5-10 ml (adult) or 3-5 ml (children) of warm water. Make sure the water flows easily.
  - Disconnect the enteral extension set. To do this, carefully turn the button connector counterclockwise until the indicator tip on the button connector lines up with the indicator line on the button. Then, carefully pull the button connector out of the button.
  - Close the feeding port on the button with the plug.
  - Thoroughly rinse the enteral extension hose set with warm water after every use. Stubborn food residues, particularly on the connectors, can be removed with a soft brush if needed. Air dry.

Note: All serious incidents occurring in connection with the devices must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or the patient is established.

**Storage**  
The products must be stored in the original packaging at room temperature in a dry environment. Avoid direct sunlight and heat.

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