ENFit®

LOW DOSE ORAL SYRINGE WITH ENFit COMPATIBLE CONNECTOR FOR ORAL AND ENTERAL USE.

INSTRUCTIONS FOR USE

DEVICE DESCRIPTION

Oral tip syringe with ENFit compatible connector.

INTENDED USE

Medicina Low Dose Oral syringes can be used for oral medicines and used for the delivery of enteral feeding, aspiration, flushing and to deliver enteral medicines to all patients groups requiring assistance for feeding. There are two types of Low Dose Oral syringe, one designed for hospital use and the second is a reusable syringe for home care patients.

INDICATIONS FOR USE

Low Dose Oral syringes are for the oral delivery of medicines and feed but also connect to gastrointestinal delivery system systems such as gastric / nasogastric devices to deliver nutritional formulations or medicine to patients physically unable to eat or swallow for various physiological conditions. It can also be used for flushing the gastric content of patients.

INTENDED USER

Medical personnel and/or appropriately trained individuals for home use.

CONTRAINDICATIONS

The syringe is not intended for parenteral use.

WARNINGS/ PRECAUTIONS

- Do not re-use or reprocess as this may lead to leakage or infection.

- The syringe is intended for use on a single patient only.

- Do not use if the packaging is open, syringe tip is visibly blocked, or the syringe is damaged.

- Should fluid get into the tip of the syringe or the moat of the Low dose syringes then it should be removed by tapping the syringe. If used with a pump system Improper selection of syringe or syringe profile in a pump menu may lead to inaccurate delivery to the patient.

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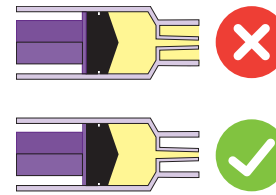
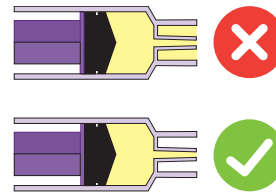
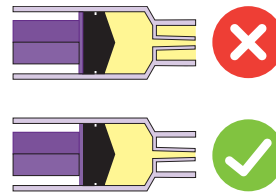
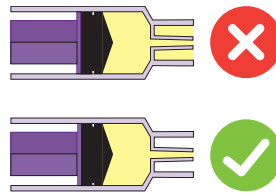
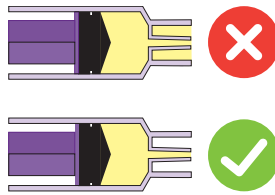
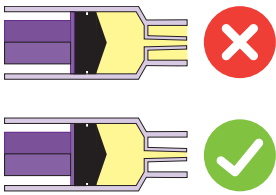
DIRECTIONS

Always follow the protocol for using Oral or Enteral syringes as advised by the medical healthcare professional.

Syringe must be connected to a male ENFit compatible connector during filling AND delivered orally or to an ENFit compatible device if delivered enterally to reduce the risk of dosing volume inaccuracies. Medicina does not recommend the use of the "cupfill" technique for filling syringes.

Fully depress the plunger before filling. Draw liquid into syringe. For accurate measurement use the leading edge of the black gasket. Do not fill beyond the maximum graduation.

To expel air bubbles, hold syringe with tip pointing up. Tap side of syringe to allow bubbles to rise toward tip. Draw back slightly on plunger, then push plunger upwards to eject air into the barrel and tip. Repeat until bubbles are eliminated.

Use immediately or cap off and label syringe until ready to use.

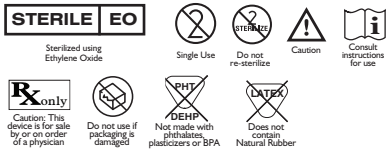
DISPOSAL

Dispose of syringe as required by Hospital Protocols. If used in home, dispose in household rubbish, or contact the local state healthcare agency

STORAGE CONDITIONS

LABELLING SYMBOLS

EC REP HMC Premedical S.p.A. Via Bosco 1/3, 41037 Mirandola (MO) – Italy

MEDICINA LTD. Unit 2-4, Rivington View Business Park, Station Road, Bolton BL6 5BN, UK. CE 1 6 3 9

ENFit is a registered trademark of GEDSA used with their permission.

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