



blunt fill and blunt filter needles

instructions for use



Medicina Ltd.
Unit 2-4 Rivington View Business Park,
Station Road, Blackrod, Bolton, BL6 5BN, UK
Telephone: 01204 695050



EC REP HMC Premedical S.p.A.
Via Bosco 1/3, 41037 Mirandola (MO) – Italy

BNIFU0021639



DEVICE DESCRIPTION:

Medicina blunt fill BN01 and Medicina blunt filter needles BN02

INDICATIONS AND INTENDED USE:

Drawing up medicines from ampoules.
BN01 for glass vials with rubber stopper and plastic ampoules
BN02 for use with glass ampoules.

INTENDED USER:

Medical professionals.

WARNINGS/ PRECAUTIONS:

A recent Adverse Event was reported whereby 'coring' could occur when using the Blunt Fill Needle with a specific Pharmaceutical drug. This specific drug was the antibiotic Piperacillin/Tazobactam. Testing has shown this not to be attributable to the Blunt Fill Needle; however please remain extra vigilant if using the Blunt Fill Needle with this drug.

CONTENT OF THE PACKAGE:

BN01 Blunt fill needle
BN02 Blunt filter needle


DIRECTIONS:

1. Secure Hub of needle to syringe by twisting clockwise. Syringe is ready to access single dose vials or multiple dose vials or ampoules.
2. Remember to puncture rubber stopper at a 90 degree angle to reduce the risk of catching the inside wall of the vial.
3. Fill syringe from single dose vial or multiple dose vial. Medicina Blunt fill filtered needle to be used with glass ampoules only. The Medicina blunt fill filtered needle contains a one way filter and should be removed after the syringe is filled.
4. Detach needle from syringe by twisting anti clockwise. Discard needle in an approved sharps container.
5. The syringe is now ready to be used with:

- Skin injections
- Luer activated valves
- IV bags
- IV systems

STORAGE CONDITIONS

Keep away from sunlight
Keep dry

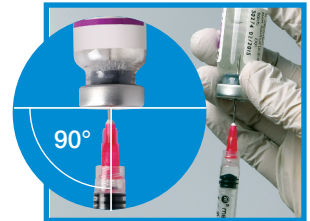

Description: BN01 and BN02 IFU			
Code: BNIFU0021639		Size: W 148mm x H 99mm (folds to 74x99mm)	
Colours: Full Colour		Date created: 17/12/2020	
Reason for artwork commission/revision: Change Logo, address, add EC representative, update symbols			
Task	Name	Signature	Date
Proofed by	Joanne Schmidt		17/12/2020
Reviewed by	Gary McMahon		17/12/2020
Approved by	Caterina Buono		20/12/2020
APPROVALSBOX001			